

Wipes Basic Bag



Disposable wipes dispenser for various B. Braun disinfectants

Product features

- Highly stable bag, easy-to-use with disinfectant level control
- PET nonwoven wipes for use with different disinfectants
- 100% PET fleece wipes that are both absorbent and lint-free consistently release the active components when disinfecting surfaces
- Ready to use in 15 minutes and lasts for six weeks – also in high-risk areas
- Reclosable lid protects against contamination and evaporation


About the product

The B. Braun Wipes Basic Bag is an innovative wipe dispenser system for hygiene procedures. When filled with medical-grade disinfectant, the system is suitable for disinfecting medical devices and surfaces of all kinds to meet medical hygiene standards. The compact B. Braun Wipes Basic Bag system is a convenient time-saver and comes prefilled with a roll of nonwoven wipes. Any disinfection procedure can be carried out after adding the disinfectant and waiting the necessary 15 minutes for it to soak in. When combined with the products listed in the table below, the B. Braun Wipes Basic Bag can remain effective for 42 days. Once the roll of wipes has been used up, the system can be disposed of immediately as there is no need for it to be reprocessed.

Spectrum of activity Use at ambient temperature (min. 20° C)	Concentrations & exposure time	
Hexaquart® XL	1%	15 min.
Hexaquart® pure	1%	5 min.
Meliseptol® rapid	conc.	1 min.
Meliseptol® Foam pure	conc.	1 min.
Meliseptol® acute	conc.	1 min.
Meliseptol® pure	conc.	1 min.
Ethanol 70% denat.	conc.	3 min.

Single-use wipe dispenser for filling with different disinfectants

Application notes

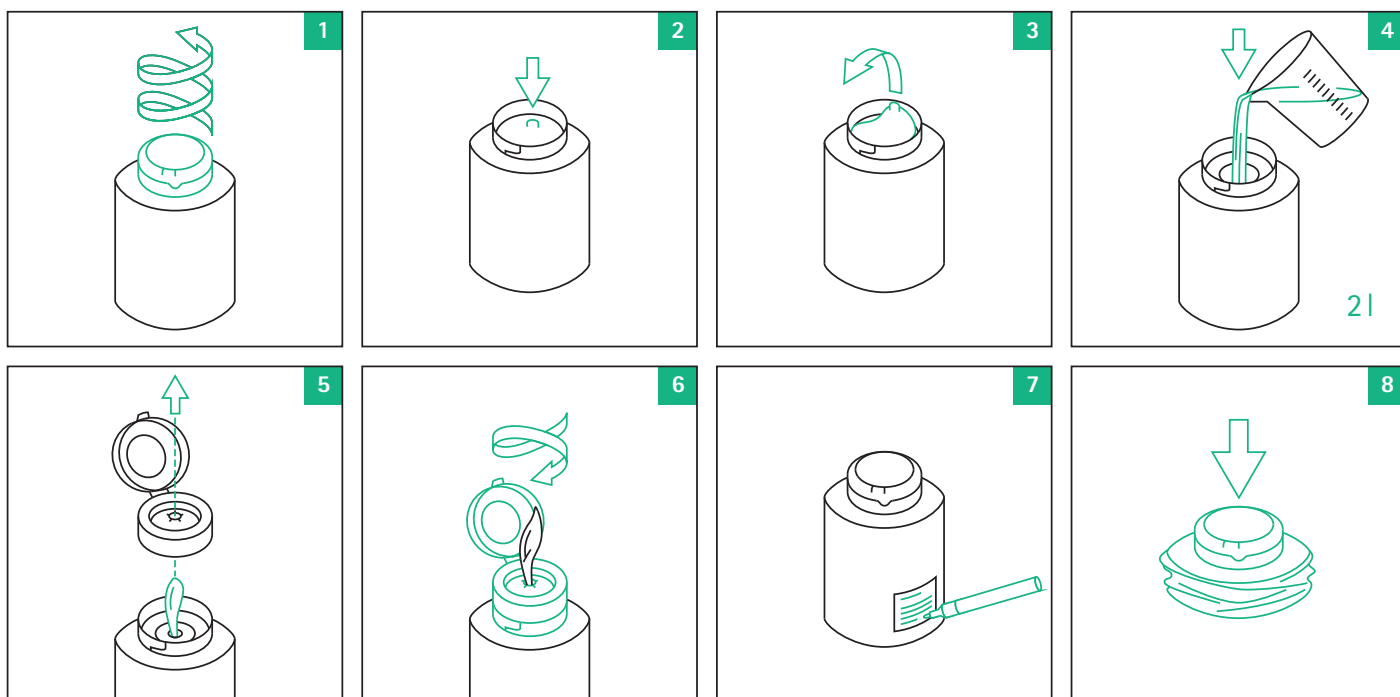

Areas of application

The B. Braun Wipes Basic Bag single-use wipe dispenser system can be used to impregnate, hygienically store and use nonwoven wipes with medical-grade surface disinfectant. These wipes can be used to clean and disinfect all kinds of medical equipment and surfaces.

User guide

B. Braun Wipes Basic Bag comes with a dry roll of nonwoven wipes already inside. You can then choose a disinfectant to pour into it:

- Prepare 2 liters of application solution for the disinfectant of your choice (check the manufacturer's information regarding the application concentration and reaction time).
- Open the protective cap on the wipe dispenser.
- Remove both parts of the membrane that make up the seal.
- Fill with 2 liters of disinfectant solution.
- Now thread the top wipe through the anchor-shaped opening in the dispenser.
- Pull out that first wipe and discard it.
- Fill out all the documentation fields on the label.

After a mere 15 minutes of presoaking, the system is ready to use. Filling instructions are available upon request.

Please contact us or the manufacturer of the disinfectant should you have any questions about the product's shelf life or suitability.

PRODUCT SIZE	REF
Basic bag with 120 wipes	180357